

Defibrillator – Monthly Check List

Cabinet

- Door lock & catch working freely
- If winter – Internal heater feels warm
- Detachable LED light works
- G5 AED is present & in its Carry Case
- Spare set of Pads are present in rear pouch of Carry Case & are in life
- Ready Kit bag is present & attached to Carry Case
- G5 AED User Guide & Performing a Rescue Sheet are present in folder

Ready Kit Contents

- Large Nitrile Gloves
- Medium Nitrile Gloves
- Disposable Razor
- Para-med 5½ inch shears
- Small Paper Towel
- Pocket Resuscitator Mask

G5 AED

- Pads are in life
- “Rescue Ready” indicator is GREEN before opening lid
- The display works & automated voice prompts begin when lid opened
- One or more Battery Level indicator LEDs are GREEN when lid opened
- “Rescue Ready” indicator returns to GREEN when lid closed

Date

Name